

The SwallowSTRONG uses a wireless internet connection (WiFi) to sync data from therapy to the SwallowSTRONG Management System (SSMS). If you have access to a WiFi connection, setting it up with your SwallowSTRONG device is quick and easy. Follow the steps below to establish the connection:



Figure 1. Home Screen. Access the WiFi Center by tapping the icon (see above). The Back and Home buttons appear in the same position on every screen.

**Step 1:** Power the device on and wait until the home screen appears. See Figure 1.

**Step 2:** Tap the 'WiFi Center' icon. See Figure 1.

**a:** A list of available networks appears.

**Step 3:** Tap the name of the network you would like to join.

**a:** If the network is password protected, a prompt asking for the password will appear.

**b:** Enter the password, and tap connect.

**c:** NOTE: If you don't know the password, contact the network administrator.

**Step 4:** Once connected, the network will have a **green checkmark** in the column marked 'Status.'

**Step 5:** Tap the **back arrow or home screen button** at the bottom of the page to return to the home screen. See Figure 1. From here tap the '**Swallow**' icon to start the SwallowSTRONG program.